HEALTH MASK

Filtering Half Mask REUSABLE

MAXIMUM TEMPERATURE: 60 °C (FIVE WASHING CYCLES ARE GUARANTEED).
FOR A LONGER LIFE OF THE PRODUCT, GENTLY HAND WASH AT 35-40 °C
 DO NOT IRON

MADE IN ITALIA

UK

MATERIALS		
OUTER LAYER	 Tested for harmful substances. www.oeko-tex.com/standard100	100% water repellent recycled polyester (110 g/m ² ± 10%)
INNER LAYER	100% Cotton (190 g/m ² ± 10%)	
ELASTICS	Spandex and Nylon	

TECHNICAL SPECIFICATIONS	
Test	Results
BREATHABILITY WITH WATER VAPOR	16,3 mg/cm ² *h
WATERPROOFNESS (outer layer)	0 mm water dragging

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Pursuant to paragraph 2 of article 16 of Legislative Decree of 17 March 2020, no. 18, issued by Italian Government

1. Wash hands and face with soap before and after using the mask and before touching it.
2. Wash mask before use and before wearing it.
3. Take the mask from the external part, put the mask on the face covering your nose, your mouth and your chin; be sure the elastics are well placed behind your head.
4. After putting the mask properly, do not touch the mask with your hands.

SIZE	One size
CODE	M031-B025 (bag containing 25 pieces)

• Pursuant to articles 102-113 of the "Consumer Code", Legislative Decree 6 September 2005, N. 206, declares that this product is neither a PPE nor a Medical Device; does not harm people's health.
• Pursuant to paragraph 2 of article 16 of Legislative Decree of 17 March 2020, no. 18, and of circular of the Ministry of Health of 18 March 2020, to contain the spread of the COVID-19 virus and until the end of the emergency state mentioned in the resolution of the Council of Ministers on January 31, 2020, individuals present on the national territory are authorized, as a precaution, to use the filter masks without the CE mark and produced notwithstanding the current marketing regulations, which for their destination are not considered neither as Medical Device nor as PPEs.
• Pursuant to paragraph 2 of article 16 of Legislative Decree of 17 March 2020, no. 18, the assessment procedures do not apply referred to in Article 15 of Legislative Decree March 17, 2020, n.18.

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Emis par Cofra S.r.l.
Emitted por Cofra S.r.l.